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
GIMA 30736

Fabbricante / Manufacturer:
SILVERFOX CORPORATION LIMITED
No. 18, 1st TongLe Road, TangXia Town, Pengjiang District,
JiangMen City, Guangdong Province, China
amy.liang@silverfox.cn
Made in China

REF 1006A

Importato da / Imported by:
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com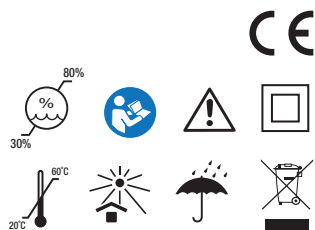

ABOUT THIS MANUAL

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

CAUTIONS



- Do not use in wet environments (like the bathroom). Clean your skin before operating.
- Turn the power off after operation.
- Please disconnect the product from power source if it not being used for a long period of time.
- Clean the lamp head and cover it properly for storage.
- Please wipe the lens with soft lens cleaning paper when cleaning the lens. Do not wipe with cloth and paper of other materials, otherwise the lens may be damaged.
- Avoid water or other liquid contact with the brightness switch when adjusting the device.
- Please avoid direct light projection to the eyes.
- If the external soft power cable is damaged, it can only be replaced by the manufacturer, service agent or personnel with similar qualification in order to avoid hazard.

- WARNING** "⚡" (IEC 60417-6042): The light source can only be replaced by the manufacturer, service agent or personnel with professional qualification.
- Do not use alcohol or other organic solvent to clean the lens.

INSTRUCTION

- This device mainly detects changes in the human skin surface by examining the various minor changes in the skin surface. Providing further effectiveness when operating beauty and care treatments.
- The magnifying function of the product can also be used for beauty treatments such as eyebrow and lips tattooing and eyelash grafting, etc.
- The magnifying lamp can be use independently with the supporting stand or in combination with other devices.
- The device has an innovative design and elegant profile. It has a safety design to avoid injury by clamping during operation. Magnifying lens is with 3D diopter, 1.75X magnification times, The brightness of the lamp is gentle and comfortable to avoid dizziness after use.

COMPONENTS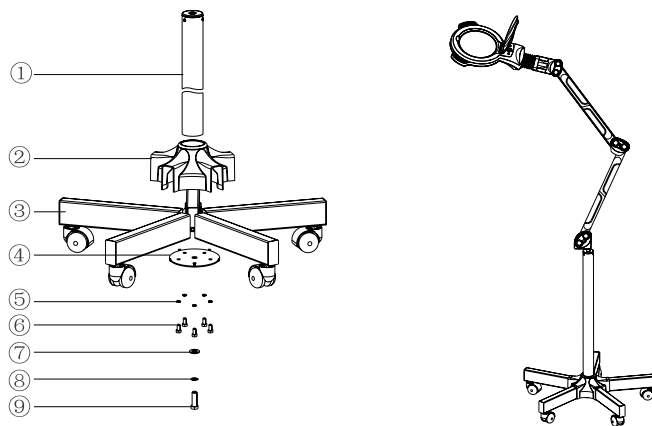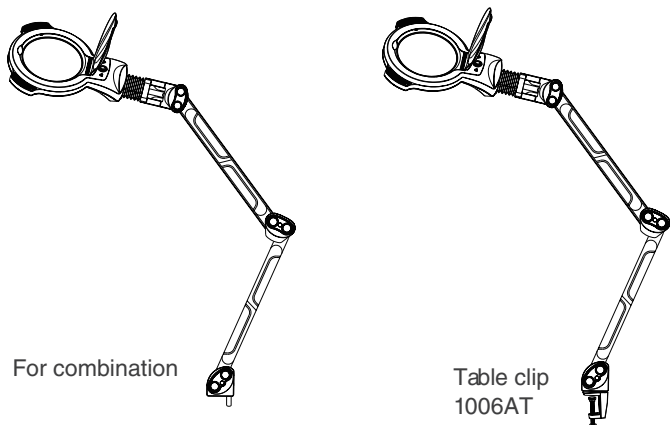

INSTALLATION PARTS

No.	Name	Qty	No.	Name	Qty
1	Column	1	6	Hexagonal screw M6X20	5
2	Pentagonal base holder	1	7	Flat pad φ10	1
3	Five-legged stand	1	8	Spring pad φ10	1
4	Pentagonal clamp plate	1	9	Hexagonal screw M10X40	1
5	Spring pad φ6	5			

OPERATION METHOD


Operation Cautions

- Warning!**
- Do not put acidic or alkaline substances in this product.
- Do not move this product when it is working.
- Do not use or store this product in an environment with acidic gas or humid air to avoid malfunction and danger.
- Do not use this product (such as alcohol, thinner, gasoline, nail polish remover, spray, etc.) near
- high-temperature and high-heat objects or near flammable objects.

Operation method

- Connect the power supply.
- Turn on the "Power" button. Open the plastic cover and align the magnifying lamp with the treatment zone.
- Adjust brightness and the distance between the lamp and skin to achieve the clearest illumination. (4 levels of brightness)
- Examine the skin or begin beauty treatment. Brightness can be adjusted during operation.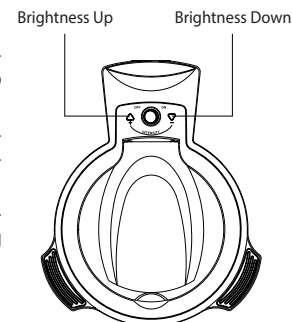

PRODUCT PARAMETERS

(1) See the table below for input power:

Rated Voltage (V)	AC 100-240V
Rated Frequency (Hz)	50/60Hz

- Rated power and illumination: (illumination tested at a distance of 20 cm)
Level1: 2.5W, 950 LUX Level2: 3.5W, 1400 LUX
Level3: 5.0W, 1900 LUX Level4: 7.0W, 2400 LUX
- Working environment: Temperature +0°C~+45°C, relative humidity 30%~90%
- Storage environment: Temperature +20°C~+60°C, relative humidity 30%~80%
- Net weight: 12.8kg
- Gross weight: 15kg
- Packing size: 83.5cm × 69.5cm × 13.5cm

INSTRUMENT MAINTENANCE

- After every treatment, please only use neutral solution to clear the dust, metal part or other parts. Disinfecting solution like alcohol, acid or alkaline solution are forbidden. This is to prevent the lifetime of the components from being shortened.
- Please lubricate the moving parts regularly (once a year)

TROUBLESHOOTING

The Fault Phenomenon	Reason	Solution
The light is not bright	1. No power. 2. The power cord is not properly inserted or connected.	1. Turn on the power. 2. Plug or connect the power cord properly.
The brightness of the light cannot be adjusted	1. No power.	1. Turn on the power.

	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
	3. Button failure	3. Contact the repair center or dealer to troubleshoot.
	4. Poor contact or burnt out of the circuit	4. Contact the repair center or dealer to troubleshoot.
Abnormal sound when the lamp arm moves	The rotating parts are short of lubricating oil	Add lubricating oil.

If problem is still not solved after above operations, please contact maintenance center or supplier to solve it.

Please check the product specification before use. This manual is for reference only, Specifications are subject to change without prior notice.

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

GIMA WARRANTY TERMS
The Gima 12-month standard B2B warranty applies.

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